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# Instructions for use

## INSERTION

- 1** "R" mark is for the right earplug.  
Position it as shown

- 2** Insert the earplug into the ear canal  
by applying a slight pressure

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**EN** User instructions  
Earplugs for sleeping

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For more information,  
please contact:

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+32 2 356 54 26  
info@decilo.be  
www.decilo.be

## REMOVAL

- 1** Position your finger on the back of  
the earplug to lift it up

- 2** Gently pull the earplug out of  
your ear

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**Here are some usage tips:** We recommend always inserting and removing the earplugs with care. Do not push or remove the earplugs with your nails. Avoid twisting the earplugs when inserting or removing them. If possible, gently remove the earplugs in a straight line.

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## EN - User guide - Earplugs for sleeping

The hearing protection is custom-made according to the ear impression. Please read the instructions for use carefully before starting to use your hearing protection.

### Insertion and removal of earplugs

The earplug marked with "R" corresponds to your right ear. Insertion: Grasp your ear protection with your thumb and index finger (using the hand on the same side as your ear) so that the "R" marking is facing forward. While keeping the marking in this position, insert the earplug into the ear canal. Then push the earplug into your ear with your index finger. During this step, you can open your mouth to make the insertion easier. Removal: Place your index finger behind the protection to lift it. Then grasp your ear protection with your thumb and index finger to remove it.

### Noise tightness

Always place the hearing protection so that the ear canal is properly blocked. Position the hearing protection correctly so that the noise reduction is guaranteed.

### Storage and Maintenance

The hearing protection is reusable. It should be cleaned after each use. Cleaning with soap and water. Chemical detergents and compressed air should be avoided as they are harmful to the product. Never heat the hearing protection to dry it. Let it dry at room temperature. The life of the hearing protection depends on the use and maintenance. We recommend storing the hearing protection in the case included in the delivery for storage and transport.

### Skin irritations and dysfunctions

The hearing protection is made of biocompatible medical silicone material. It is coated with a hypoallergenic lacquer. Consult an ENT specialist in case you still have itching or allergies. This may be an over-reaction of the skin against the material used. Please contact your dealer or the manufacturer in the event of incorrect positioning of the protection or if the sealing effect of your hearing protection is insufficient. If the hearing protection is inserted incorrectly or if the instructions for use are not followed, the effectiveness of the hearing protection may be impaired.

### Warnings

The hearing protection is for personal use only and should not be worn by another person. The retailer and the manufacturer accept no liability for a possible reduction in noise tightness. They also decline all responsibility for the consequences of wearing hearing protection. The hearing protection cannot be used as part of a noise protection system. Hearing protection is not designed to protect the ear from water. Sleep hearing protection is considered to be comfort hearing protection and is not subject to personal protective equipment certification. Contact the manufacturer or your dealer for further information. It is the responsibility of the user to check the suitability for the intended use, taking into account any influences that may affect performance. Warranty and liability claims for our products are provided for in the general terms and conditions of sale, unless otherwise stipulated by law.