FR

Mode d'emploi

Protection auditive sur mesure

NL

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INSERTION

1

"R" mark is for the right earplug.
Position it as shown

2

Insert the earplug into the ear canal
by applying a slight pressure

REMOVAL

1

Position your finger at the bottom
of the ear canal as shown

2

Gently push your finger so that the
earplug comes out of your ear

If your earplugs have a cord, **NEVER** remove them by pulling on the cord as this can damage the product

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H-high frequency attenuation (f>2000 Hz); M-medium frequency attenuation (f>500 Hz and f<2000 Hz); L-low frequency attenuation (f<500 Hz); SNR-Single Number Rating; m-average value; s-standard deviation

Modèles decilo Soft & Hard certifiés par Alienor Certification à 86100 Châtellerault (2754) / decilo Soft & Hard modellen gecertificeerd door Alienor Certification in 86100 Châtellerault (2754) / decilo Soft & Hard models certified by Alienor Certification at 86100 Châtellerault (2754) / decilo Soft & Hard Modelle zertifiziert durch Alienor Certification in 86100 Châtellerault (2754)

Ces bouchons d'oreille ont satisfait aux exigences de l'essai de résistance aux chutes à -20°C / Deze oordopjes voldoen aan de eisen van de valproef van -20°C / These earplugs have met the requirements of the -20°C drop test / Diese Ohrstöpsel haben die Anforderungen des Falltests bei -20°C erfüllt.

Remark: decilo Soft Lin. Ocean Model - No Personal Protective Equipment - SNR: 10 dB H: 10 dB M: 6 dB L: 6 dB

Frequency (Hz)		125	250	500	1000	2000	4000	8000
decilo® Soft	decilo Soft Lin. Blue	SNR: 15 dB H: 14 dB M: 13 dB L: 14 dB SNRm: 17 dB Hm: 16 dB Mm: 15 dB Lm: 16 dB SNRs: 2 dB Hs: 2 dB Ms: 2 dB Ls: 2 dB						
	Mean Attenuation (dB)	19.3	17.5	17.0	15.2	15.5	17.9	18.7
	Standard Deviation (dB)	4.7	3.2	4.4	2.2	2.9	3.7	5.6
	Assumed Protection Value APV (dB)	14.6	14.4	12.6	12.9	12.6	14.1	13.2
	decilo Soft Lin. White	SNR: 17 dB H: 14 dB M: 17 dB L: 18 dB SNRm: 19 dB Hm: 17 dB Mm: 19 dB Lm: 21 dB SNRs: 2 dB Hs: 3 dB Ms: 2 dB Ls: 2 dB						
	Mean Attenuation (dB)	24.3	22.1	23.7	22.2	17.7	17.8	17.7
	Standard Deviation (dB)	3.9	3.8	3.5	3.6	3.6	3.1	5.3
	Assumed Protection Value APV (dB)	20.4	18.2	20.2	18.7	14.1	14.7	12.4
	decilo Soft Lin. Black	SNR: 21 dB H: 19 dB M: 20 dB L: 20 dB SNRm: 23 dB Hm: 21 dB Mm: 22 dB Lm: 22 dB SNRs: 2 dB Hs: 2 dB Ms: 2 dB Ls: 2 dB						
	Mean Attenuation (dB)	24.6	23.0	24.3	22.7	21.2	19.9	35.5
	Standard Deviation (dB)	3.9	4.4	4.5	1.9	3.0	2.6	6.1
	Assumed Protection Value APV (dB)	20.7	18.6	19.8	20.8	18.3	17.4	29.4
decilo® Hard	decilo Soft N-Lin. Yellow	SNR: 24 dB H: 24 dB M: 22 dB L: 21 dB SNRm: 26 dB Hm: 26 dB Mm: 24 dB Lm: 23 dB SNRs: 2 dB Hs: 2 dB Ms: 2 dB Ls: 2 dB						
	Mean Attenuation (dB)	21.0	22.2	24.5	23.9	28.3	24.4	31.6
	Standard Deviation (dB)	4.5	3.6	2.4	2.7	3.2	2.8	5.5
	Assumed Protection Value APV (dB)	16.4	18.6	22.1	21.1	25.1	21.6	26.1
	decilo Soft N-Lin. Purple	SNR: 26 dB H: 25 dB M: 23 dB L: 23 dB SNRm: 27 dB Hm: 27 dB Mm: 26 dB Lm: 26 dB SNRs: 2 dB Hs: 2 dB Ms: 2 dB Ls: 3 dB						
	Mean Attenuation (dB)	26.7	26.3	27.1	24.7	30.3	25.6	43.5
	Standard Deviation (dB)	4.9	3.2	4.2	2.9	3.9	3.0	5.0
	Assumed Protection Value APV (dB)	21.8	23.1	22.9	21.9	26.3	22.6	38.5
	decilo Hard N-Lin. Yellow	SNR: 25 dB H: 25 dB M: 24 dB L: 21 dB SNRm: 27 dB Hm: 27 dB Mm: 26 dB Lm: 23 dB SNRs: 2 dB Hs: 2 dB Ms: 2 dB Ls: 2 dB						
	Mean Attenuation (dB)	20.2	21.5	25.1	26.5	30.2	25.9	30.1
	Standard Deviation (dB)	3.5	3.2	3.8	2.2	3.2	2.6	5.2
	Assumed Protection Value APV (dB)	16.7	18.3	21.3	24.2	27.0	23.3	24.9
	decilo Hard N-Lin. Purple	SNR: 31 dB H: 30 dB M: 29 dB L: 26 dB SNRm: 32 dB Hm: 32 dB Mm: 31 dB Lm: 29 dB SNRs: 1 dB Hs: 2 dB Ms: 2 dB Ls: 3 dB						
	Mean Attenuation (dB)	28.4	26.8	30.8	31.0	34.0	31.4	39.5
	Standard Deviation (dB)	6.1	4.0	2.5	2.9	3.2	3.5	5.4
	Assumed Protection Value APV (dB)	22.3	22.8	28.2	28.1	30.8	28.0	34.1

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Optionele accessoires

Reinigingstool, Filter Tool, Set Filters, Handgreep, Koord, Kledingclip

EN	User instructions
----	-------------------

Hearing protection is custom made according to your ear impressions. It will comfortably protect your ears from dangerous noise levels. Please read the instructions for use carefully before starting to use your hearing protection.

Use

Before each use, make sure the hearing protection is clean. The earplug with the "R" mark corresponds to your right ear. Grasp your hearing protection with your thumb and index finger (with your hand on the same side of your ear) so that the "R" marking is facing forward. Keeping the marking in this position, insert the earplug into the ear canal. Then push the earplug into your ear using your index finger. During this step, you can open your mouth to make it easier to insert the earplug. To remove your hearing protection, position your index finger at the bottom of the ear canal between your tragus and antitragus. Gently push your index finger so that the protection comes out of the ear. Then grasp your hearing protection with your index finger and thumb. **Warning: If your hearing protection has a cord, never remove the hearing protection from you ear by pulling on the cord.**

Cleaning & Storage

The hearing protection is reusable. Your earplugs can be easily cleaned after each use with water and soap using a humid wipe. Let your earplugs air dry at room temperature. NEVER heat the hearing protection. Some chemicals may have a negative effect on this product. Please ask the manufacturer for more information. The lifetime of the hearing protection depends on its use and care. After use, we recommend storing the hearing protection in the box provided. We invite you to remove any earwax that may be in the canal of the earplug by using the wax removal tool provided. This step should be done carefully to avoid touching the acoustic filter or damaging it. NEVER touch the filter with sharp objects as this may damage the acoustic membranes of the filter. The filter should generally not be cleaned. If deep cleaning is necessary, remove the filter from the earplug and shake it in water. Then let it air dry at room temperature. Failure to follow these recommendations may result in damage to your hearing protection.

Warnings

In case of misuse, the hearing protection will not provide sufficient protection against noise. In noisy environments, we recommend that you wear your hearing protection at all times. Removing your hearing protection, even for a very short period of time, will considerably reduce its effectiveness. Contact your supplier in case of insufficient adaptation or if the sealing of your hearing protection is inadequate. Warning: Do not use these earplugs if there is a risk that the connection element may be caught in the course of its use. Warning: Sudden or rapid removal of the earplugs from the ear canal can damage the eardrum. The hearing protection is intended for personal use only and should not be worn by another person under any circumstances. Your hearing protection should be checked for leaks once a year. We recommend that you change your hearing protection every four years. The greatest care is taken in the manufacturing of your earplugs. The hearing protection is made of hypoallergenic and biocompatible medical grade material. If you notice an abnormality, please remove the hearing protection, contact your supplier and, if necessary, inform your doctor. Warranty and liability claims for our products are provided in the general terms and conditions of sale.

Declaration of conformity

decilo SRL hereby declares that the hearing protection is in compliance with European PPE Regulation (EU) 2016/425. The quality of the earplugs is tested & certified according to the EN 352-2 standard, by notified body Alienor Certification, 21 Rue Albert Einstein, 86100 Châtelleraut, France with identification number 2754. The declaration of conformity can be found on www.decilo.be (Downloads).

Optional accessories

Cleaning Tool, Filter Tool, Filters Kit, Handle, Cord, Shirt Clip

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